



Owner's Booklet



FreeStyle

InsuLinx

Blood Glucose Monitoring System



 **Abbott**

FreeStyle InsuLinx at a Glance

Touch screen

Displays your test results and other important information. Use touch screen buttons to work with your meter.

USB port

Plug in your micro USB cable here to download test results.

Test strip port and light

Insert a test strip here. The meter powers on when you insert a test strip.

Meter Home button

This button is used to turn the meter on and off, control the backlight and to go back to the Home screen.

Test strip

Insert strip into the meter with printed side up.

Sample areas

Apply blood or control solution to one sample area only.




















Table of Contents

Symbols	1
Important Information About Your Blood Glucose Monitoring System	2
Intended Use	3
Set up Your Meter for the First Time	4
Getting to Know Your Meter	6
Turn Your Meter On and Off	6
Find the Home Screen	6
Use the “?” Help Button	7
Turn the Backlight On and Off	7
Turn the Strip Light On and Off	8
Test Your Blood Glucose	9
Your Test Results	14
Low and High Blood Glucose Results	15
Use Meal Markers	17
Error Messages	18
Temperature Warning	18
Control Solution Test	19
View Logbook	22
Change Your Meter Settings	24
Personalize Your Meter	25
Set up and Use Weekly Messages	26
Set up Weekly Messages	26
Use Weekly Messages	27

Set up and Use Reminders	28
Set up Reminders	28
Use Reminders	29
Taking Care of Your Meter	30
Changing Batteries	30
Cleaning and Disinfecting Your Meter	31
Troubleshooting	33
FreeStyle Auto-Assist: Progress Reporting Software	35
Getting to Know Your FreeStyle Auto-Assist Software	35
Install the FreeStyle Auto-Assist Software	36
FreeStyle Auto-Assist System Requirements	38
Change Your Meter Settings	38
Set up Advanced Features for Your Meter Logbook	39
Set up the Insulin Log Feature	39
Set up Notes for Your Meter Logbook	40
Set up Meter Reminders	41
Set up Your Meter Personalization	42
Change Rapid-acting Insulin Dose Increments	43
Set up Weekly Messages on Your Meter	44
FreeStyle Auto-Assist Reports	45
Set up or Change Report Preferences	47
Create, View, Save, and Print Reports	48
Additional FreeStyle Auto-Assist Features	49
Meter Specifications	50
Appendix — End of Meter Life/ Backup Battery Removal	52

Symbols

Symbol	What It Means	Symbol	What It Means
	Settings		View blood glucose results in chart form when weekly messages option is turned on
	Extras		Drop down menu for sorting results in logbook chart view
	Strip light on/off switch		Meal marker: Before meal result
	Reminders		Meal marker: After meal result
	Logbook		Meal marker: Other time/no mark
	Insulin marker		Indicates a control solution test result
	Weekly messages (not read and read)		Low battery
	Insulin log feature		Caution
	Meter out of temperature range		

Important Information About Your Blood Glucose Monitoring System

CAUTION:

- Please read all the instructions provided in this Owner's Booklet and practice the testing procedures before using the FreeStyle InsuLinx system. Before setting up and using your meter, work with your health care professional to determine which meter features are most appropriate for you and your diabetes management.
- The meter and its accessories are for use by a single person. They must not be used on more than one person including other family members due to the risk of spreading infection. Do not use on multiple patients! All parts of the meter and its accessories are considered biohazardous and can potentially transmit infectious diseases, even after performing the cleaning and disinfection procedure.^{1,2}
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be disinfected prior to use by the second person.
- Not for use on patients who are dehydrated, hypotensive, in shock, or for individuals in hyperglycemic-hyperosmolar state, with or without ketosis.
- Critically ill patients should not be tested with a blood glucose meter.

WARNING: This system contains small parts that may be dangerous if swallowed.

When removing your new FreeStyle InsuLinx system from its packaging, check that the contents are undamaged and that you have all parts listed below. If any parts are missing or damaged, contact Customer Service.

Your kit contains the following contents:

- 1 FreeStyle InsuLinx Meter
- 1 Micro USB Cable
- 1 Owner's Booklet
- 1 Carrying Case
- 1 FreeStyle Lancing Device
- 10 Lancets
- 10 FreeStyle InsuLinx Test Strips (may be sold separately)
- 1 FreeStyle Control Solution - Normal (may be provided separately)

FreeStyle Control Solution is available in three ranges - Low, Normal and High. For additional supplies, please contact Customer Service at 1-800-222-3100 or your distributor or retail pharmacy.

Intended Use





The FreeStyle InsuLinx Blood Glucose Monitoring System is intended for the quantitative measurement of glucose in fresh capillary whole blood samples drawn from the fingertip. The FreeStyle InsuLinx Blood Glucose Monitoring System is intended to be used by a single person and should not be shared.

The FreeStyle InsuLinx Blood Glucose Monitoring System is intended for self testing outside the body (*in vitro* diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The FreeStyle InsuLinx Blood Glucose Monitoring System should not be used for the diagnosis of or screening for diabetes or for neonatal use.

The FreeStyle InsuLinx Blood Glucose Test Strips are for use with the FreeStyle InsuLinx Blood Glucose Meter to quantitatively measure glucose in capillary whole blood samples drawn from the fingertip.

Set up Your Meter for the First Time

Set the language, current date, and time.

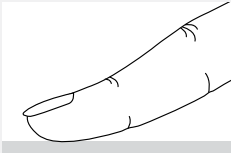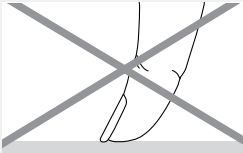
Step	Action	
1		Press the meter Home button.
2		<p>Tap the language you want the meter to use.</p>   <p>Use the pad of your finger.</p> <p>Do not use your fingernail or any other object to tap the screen</p> <p>Note: The OK button will appear when you select the language. Tap the OK button to move to the next screen.</p>

3



Tap the **arrow buttons** to change the date. Then tap **save**.


4

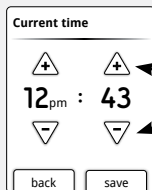

Tap the **arrow buttons** to set the time. Then tap **save**.

Notes:

- You will see the Home screen when you complete this step.

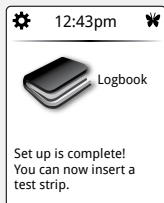

- If you don't complete these steps, the meter will continue to prompt you to set the language when you turn it on.

IMPORTANT: If the date and time are not set correctly, some meter functions may not work properly. If the meter loses time, the home screen will display --:-- until it is reset.

Getting to Know Your Meter

Learn how to turn your meter on and off, find the Home screen, use the ? Help button, and use the backlight and strip light.

Turn Your Meter On and Off



To turn the meter on:

- Press the meter Home button, or
- Insert a test strip.

To turn the meter off:

- If you're on the home screen, press the meter Home button, or
- Remove a test strip, or
- Do nothing for **2 minutes**.


Find the Home Screen



Most of your meter activities will begin at the Home screen.

Press the meter Home button anytime to return to the Home screen **except** when a test strip is inserted.

Use the “?” Help Button

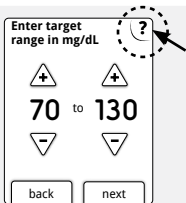

Tap the ? Help button anytime you see it and want more information.

Turn the Backlight On and Off

The backlight can help you view the screen in low light.

To turn the backlight on:

- Press and hold the meter Home button for **3 seconds**.


To turn the backlight off:

- Wait for 15 seconds, or
- Press and hold the meter Home button for **3 seconds**.


Turn the Strip Light On and Off



To turn the strip light on:

- If a test strip is inserted, press and hold the meter Home button for **3 seconds**.
- If a test strip is not inserted, turn the backlight on, then tap the strip light symbol  on the home screen.

To turn the strip light off:

- Tap the strip light symbol .

Test Your Blood Glucose



Get a blood glucose test result.

You will need:


- FreeStyle InsuLinx test strips that are not expired.
- Lancing device and new lancets to obtain a blood sample.

IMPORTANT:

- Use only FreeStyle InsuLinx test strips. Only use a test strip once.
- Read the test strip insert before performing your first blood glucose test. It contains important information and will tell you how to store and handle the test strips.
- The meter and its accessories are for use by a single person. They must not be used on more than one person including other family members due to the risk of spreading infection. All parts of the meter and its accessories are considered biohazardous and can potentially transmit infectious diseases, even after performing the cleaning and disinfection procedure.^{1,2}
- Do not re-use lancets due to risk of infection. Not suitable for re-sterilization. New lancets can be obtained at a retail pharmacy.
- Bruising may occur at the test site. If bruising occurs, consider selecting another site.
- Refer to the lancing device insert for detailed instructions on how to use the FreeStyle lancing device.








Step	Action	
1		<p>Wash your hands with soap and warm water. Rinse and dry thoroughly.</p> <ul style="list-style-type: none">• Do not use lotion or cream on the test site.
2		<p>Check the test strip expiration date.</p> <ul style="list-style-type: none">• Do not use expired test strips; they may cause inaccurate results.

3

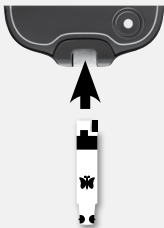

Perform the test.

- Insert a test strip and close the test strip vial.
- Use your lancing device loaded with a lancet to obtain a blood drop from your fingertip.

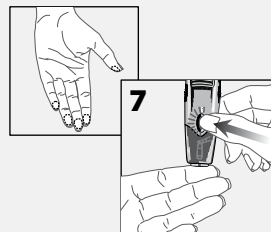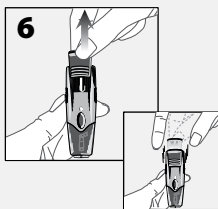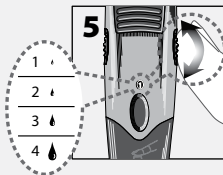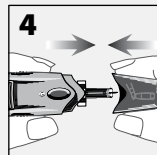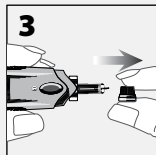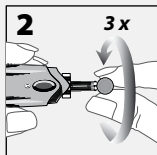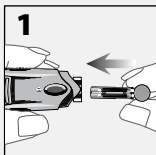

Continued on next page

3


(Continued)



- c. Apply blood to the test strip as shown in the picture.
- d. The meter will beep once to let you know that you have applied enough blood.

Notes:

- Once you start applying blood, you have 60 seconds to fill the test strip on the same side of the test strip only. The time you have left is shown as a countdown on the screen.
- The time that the meter takes to display a result depends on your blood glucose level. The higher your blood glucose level, the longer it takes to get a result.
- The meter turns off after 2 minutes of inactivity. Remove and reinsert the unused test strip to restart the meter.

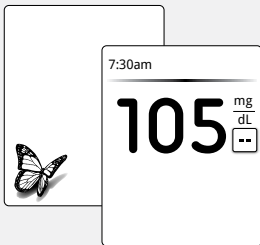

View the test result.

- A butterfly will appear on the screen while you wait for your result. The meter will beep twice when your result is ready.
- Remove and properly discard the test strip and lancet. Discard your used lancet properly in a puncture resistant container.

Refer to the *Your Test Results* section to learn how to understand and mark your test results.

Notes:

- Your result is now stored in your logbook.
- You should clean and disinfect your meter at least once per week. Refer to the *Cleaning and Disinfecting Your Meter* section for instructions.

IMPORTANT: After performing a blood glucose test, wash your hands with soap and water. Thoroughly dry your hands.



Your Test Results

View, understand, and mark your test results; know what to do if you receive an error message.

IMPORTANT:

- The meter displays results from 20 mg/dL to 500 mg/dL. Low or high blood glucose results can indicate a potentially serious medical condition. Contact your health care professional before taking any action based on an unclear message or result.
- The expected blood glucose level for a non-diabetic, non-pregnant fasting adult is under 100 mg/dL.³
- Two hours after meals, levels should be less than 140 mg/dL.³
- Consult your health care professional to determine the blood glucose levels that are appropriate for you.
- Consult your health care professional before changing your diabetes management program.

Low and High Blood Glucose Results

Display	What To Do
<p data-bbox="111 156 316 215"> Low glucose</p> <p data-bbox="107 246 282 373">Appears when result is: lower than 70 mg/dL</p>	<p data-bbox="351 151 1204 177">Follow your health care professional's advice to treat low blood glucose.</p>
<p data-bbox="111 487 316 547"> High glucose</p> <p data-bbox="107 578 282 705">Appears when result is: higher than 240 mg/dL</p>	<p data-bbox="351 479 1215 505">Follow your health care professional's advice to treat high blood glucose.</p>



Display	What To Do
<p data-bbox="111 107 207 166">LO</p> <p data-bbox="83 192 257 325">Appears when result is: lower than 20 mg/dL</p>	<ol data-bbox="299 107 1226 240" style="list-style-type: none"><li data-bbox="299 107 1226 166">1. LO may indicate that your blood glucose is very low. Follow your health care professional's advice immediately and then repeat the test.<li data-bbox="299 177 1226 240">2. If your blood glucose results do not match how you feel, perform a control solution test to confirm that your meter and strips are working properly.
<p data-bbox="111 397 200 456">HI</p> <p data-bbox="83 493 257 625">Appears when result is: higher than 500 mg/dL</p>	<ol data-bbox="299 387 1269 565" style="list-style-type: none"><li data-bbox="299 387 1269 482">1. HI may indicate that your blood glucose is very high. Wash your hands and then repeat the test. If the result is HI again, follow your health care professional's advice immediately.<li data-bbox="299 503 1269 565">2. If your blood glucose results do not match how you feel, perform a control solution test to confirm that your meter and strips are working properly.

IMPORTANT:


- Severe dehydration may cause false low results. If you believe you are suffering from severe dehydration, contact your health care professional immediately.
- Contact your health care professional if you have symptoms that do **not** match your test result, and you have followed the instructions in this Owner's Booklet.

Use Meal Markers

Mark your blood glucose test result as 'Before meal' or 'After meal', or as another time. Meal markers are used to provide additional information for Weekly Messages and in the FreeStyle Auto-Assist reports.

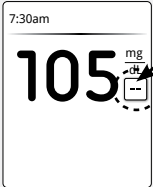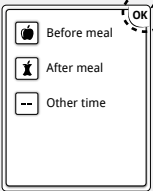Step	Action
1	 <p>When viewing your blood glucose test result, tap the dashes -- .</p>
2	 <p>Tap your selection. Tap OK to return to the result screen.</p> <p>Notes:</p> <ul style="list-style-type: none">• If you mark as Before meal, you can choose to set a test reminder for 1 or 2 hours after your meal.• You can select Other time if you are testing at times other than before or after a meal.• If you need to, you can change a meal marker for the current result by tapping the meal marker button from the result screen.

Error Messages

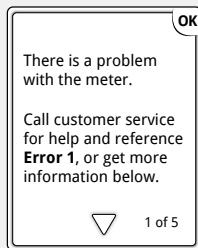

Example of an Error Message

Your meter will explain what an error message means, its possible causes, and what to do about it.

Notes:

- Remember to tap the arrow button to see all of the important information.
- Tap **OK** to return to the Home screen.

Temperature Warning



This symbol may appear after inserting a test strip. It will include a message saying the meter is either too hot or too cold to use right now. Blood glucose tests taken outside of the operating temperature range may not be accurate.



Tap **OK** to clear the screen and test again later.

Control Solution Test

A control solution test should be performed when you are not sure of your results and want to confirm that your meter and test strips are working properly.

IMPORTANT:


- Control solution results should fall within the control solution range printed on the test strip vial.
- Do **not** use control solution past its expiration date. Discard control solution 3 months after opening or on the expiration date printed on the bottle, whichever comes first. (Example: open April 15, discard July 15; write the discard date on the side of the bottle.)
- The control solution range is a target range for control solution only, not for your blood glucose level.
- Use only FreeStyle control solution with this meter.
- Replace the cap securely on the bottle immediately after use.
- Do not add water or other liquid to control solution.
- Contact Customer Service for information on how to obtain control solution.

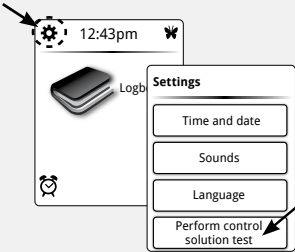Step	Action
<p>1</p>	<p>At the Home screen, tap Settings ⚙️. Then tap Perform control solution test.</p>  <p>The image shows a portion of an Android home screen. At the top, there is a status bar with a gear icon, the time '12:43pm', and a butterfly icon. Below the status bar is a dock with a 'Logb' icon. A 'Settings' app icon is visible, and a Settings menu is open over it. The menu items are 'Time and date', 'Sounds', 'Language', and 'Perform control solution test'. An arrow points to the 'Perform control solution test' option.</p>
<p>2</p>	<p>Perform the control solution test.</p> <ol style="list-style-type: none"> Insert a test strip and close the test strip vial. Apply control solution to the test strip as shown in the picture. The meter will beep once to let you know that you have applied enough control solution. A butterfly will appear on the screen while you wait for the result. The meter will beep twice when the result is ready.  <p>The image is a close-up illustration of a hand holding a small vial of control solution. The hand is using a pipette to apply a drop of the solution onto a test strip that is inserted into a device. The text 'Apply control solution to strip' is written above the illustration.</p>

3

Compare the control solution result to the range printed on the test strip vial. The result should fall within the range.

Notes:




- If you see an error message on the screen, your meter will explain what it means, its possible causes, and what to do next.
- The result will be marked as a control solution test in your logbook. It will **not** be included in your charts and reports.
- You may also mark a control solution test after it is performed by following these steps:
 - a. At the Home screen, tap **Logbook** .
 - b. Tap your most recent test. You can only modify this result.
 - c. Tap the down arrow. Then tap **add or edit notes**.
 - d. Tap the option for **control solution test**. Then tap **OK**.
 - e. You can now see your most recent result marked as a control solution test.

Out of Range Control Solution Results

- Repeat the test if control solution results are outside the range printed on the test strip vial.
- Stop using the meter if control solution results are consistently outside the range printed on the test strip vial. Contact Customer Service.

View Logbook

Review blood glucose history; review and log meals, insulin doses, notes about your test results, and control solution tests.

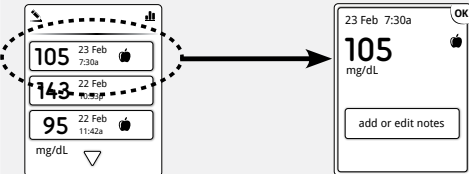

Step	Action
1	<p>At the Home screen, tap Logbook .</p> 
2	<p>Review your results.</p> <p>Notes:</p> <ul style="list-style-type: none">• Use the down arrow to scroll through logbook entries.• Tap any entry to see details of results and notes, if any. 

Continued on next page

2



(Continued)

Notes (Continued):

- If you turned on the Notes feature through FreeStyle Auto-Assist, you can tap the **add or edit notes** button to add or edit notes to your most recent result. Refer to the FreeStyle Auto-Assist *Set up Advanced Features for Your Meter Logbook* section for how to set up this feature.
- You can tap the  at the top left of the screen to use the Insulin Log feature and enter your insulin dose. Refer to the FreeStyle Auto-Assist *Set up Advanced Features for Your Meter Logbook* section for how to turn off this feature.
- If you turned on Weekly Messages, you will see the chart symbol  at the top right of the screen. Tap it to review your results for up to 14 days. To learn more about weekly messages, refer to the *Set up and Use Weekly Messages* section.
- A LO result (under 20 mg/dL) is included in the memory as LO.
- A HI result (over 500 mg/dL) is included in the memory as HI.


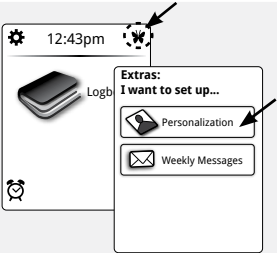

Change Your Meter Settings

Change time and date, sounds, or language.

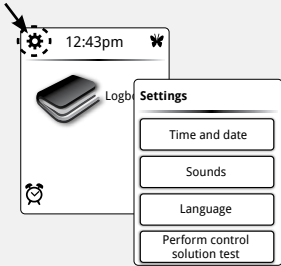Step	Action
1	 <p>At the Home screen, tap Settings  . Then tap the setting you want to change.</p> <p>Note: Tap Sounds to change Notification and Button sounds.</p> <ul style="list-style-type: none">• Notification sound On: Meter will beep when the test strip is filled, the result appears on the screen, an error occurs, and when you get a reminder.• Button sound On: Meter will beep each time you tap a button.
2	<p>Press the meter Home button to return to the Home screen.</p> <p>Note: You can change your meter settings anytime by simply repeating Step 1.</p>

Personalize Your Meter

Change the picture you see when you perform a blood glucose test.



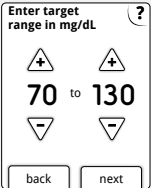
Step	Action
1	<p>At the Home screen, tap Extras  . Then tap Personalization.</p>  <p>The screenshot shows the Home screen with a clock at 12:43pm, a Logbook icon, and an Extras icon. A menu is open over the Extras icon, showing options for Personalization and Weekly Messages. Arrows point to the Extras icon and the Personalization option.</p>
2	<p>Select your picture and tap yes to confirm the change.</p> <p>Notes:</p> <ul style="list-style-type: none">• You can also use my picture to select a personal picture such as a family member or pet. Refer to the <i>FreeStyle Auto-Assist Set up Your Meter Personalization</i> section for how to upload your picture.• You can change your picture anytime by simply repeating Steps 1 and 2.  <p>The screenshot shows a 'Pictures' selection screen with a question mark icon. It contains several options: 'my picture', a photo of a baby, a photo of a dog, 'none', 'cancel', and 'more'.</p>

Set up and Use Weekly Messages

Turn on and use weekly messages. This feature will help you identify patterns in your blood glucose results.

IMPORTANT: Ask your health care professional for your blood glucose target range.



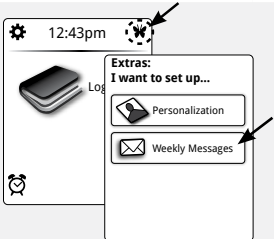


Set up Weekly Messages

Step	Action
1	 A screenshot of a mobile device's home screen. At the top, the time is 12:43pm. There are icons for settings, a calendar, and a clock. A 'Log' icon is also visible. A 'Extras' menu is open, showing options for 'Personalization' and 'Weekly Messages'. Arrows point to the 'Extras' icon on the home screen and the 'Weekly Messages' option in the menu. <p>At the Home screen, tap Extras .</p> <p>Then tap Weekly Messages.</p>
2	 A screenshot of a screen titled 'Enter target range in mg/dL'. It features a large '70 to 130' in the center. Above and below the numbers are plus and minus signs with arrows, indicating they can be adjusted. There are 'back' and 'next' buttons at the bottom. A question mark icon is in the top right corner. <p>Enter your target range. This is the desired target range for blood glucose values. This target range will only be used to determine Weekly Messages. Tap next to save.</p> <p>Notes:</p> <ul style="list-style-type: none">• Target values cannot be set below 70 mg/dL or above 180 mg/dL.• Tap the ? Help button for more information.

3 Tap **done** to return to the Home screen.

Note: You can change your weekly messages settings anytime by simply repeating the setup procedure.



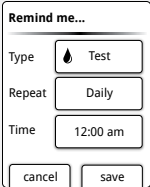
Use Weekly Messages

Step	Action
1	Perform a blood glucose test.
2	<p>Mark your meal during the blood glucose test.</p> <p>If you have set up weekly messages, you will see this screen while you wait for your blood glucose test result. Tap your selection.</p> <div data-bbox="248 373 426 598"><p>This test is...</p><p> Before meal</p><p> After meal</p><p>-- Other time</p></div>
3	<p>Tap  Message to read weekly messages from the results screen. Then tap OK to get back to your blood glucose test result.</p> <p>Notes:</p> <ul style="list-style-type: none">• Weekly messages will also be stored in your logbook.•  is displayed after the message is read. <div data-bbox="234 664 422 897"></div>


Set up and Use Reminders

Set your meter to remind you when to test, when to take your insulin, or for other alarms.

Set up Reminders

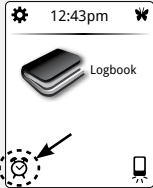
Step	Action
1	 <p>At the Home screen, tap the Clock .</p> <p>You will see the 'Remind me' screen with your menu of options.</p>
2	 <p>a. Tap Type of reminder button to select which reminder you want to set (Test, Take Insulin, or Alarm).</p> <p>b. Tap Repeat button to set how often you want the reminders (Daily, Once, Countdown Timer).</p> <p>c. Tap Time and tap the arrow buttons to set the reminder time.</p> <p>Tap save.</p>

3

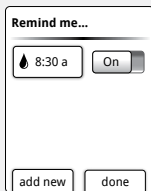

You can now:


- Tap the **On/Off** button to turn a reminder on or off.
- Tap **add new** to set more reminders.

Tap **done** to return to the Home screen. When any reminders are turned On, the time of your next reminder will appear on your home screen next to the .

Notes:

- You can change your reminders anytime by simply repeating the setup procedure.
- To add a reminder such as when to exercise, refer to the *FreeStyle Auto-Assist Set up Meter Reminders* section.

Use Reminders

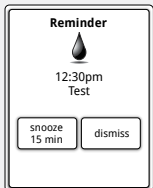

Your reminder will appear even if the meter is turned off.

Tap your selection to **snooze** or **dismiss** your reminder.

Notes:



- Missed reminders will remain on the screen until dismissed.
- If you have set an insulin reminder, you will be able to log a dose after tapping **dismiss** when the reminder appears. This will not work if you turn the Insulin Log feature off. If you enter a rapid-acting insulin dose here, it will be stored as "meal" insulin when you print reports through FreeStyle Auto-Assist.

Taking Care of Your Meter

Take care of your meter and keep it operating safely.

Changing Batteries

Note: The meter uses **2** replaceable CR 2032 coin cell batteries.

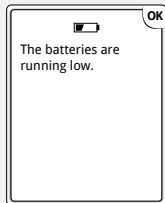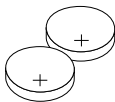


You will see this screen when your batteries are low.

Note: Your meter settings and logbook information will be saved when you change the batteries.

IMPORTANT: After you first see this warning, you can perform approximately 28 tests before you need to change the batteries.

Step	Action
1	<p>Change your batteries.</p> <ol style="list-style-type: none">Slide open the battery door on the back of the meter.Remove the old batteries.Install new batteries with (+) facing up.Slide the door back into place until it clicks.

Note: The next time you turn your meter on, it will ask you to confirm the time and date.

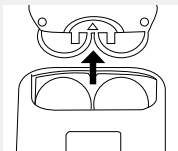

Used batteries should be removed and disposed as required by local regulations.

Cleaning and Disinfecting Your Meter and Lancing Device

Cleaning and disinfecting your meter and lancing device is important to prevent the spread of infectious diseases. It is recommended that you clean and disinfect your meter and lancing device a minimum of once per week. The meter and lancing device have a mean service life of 5 years, which is 522 cleaning and 522 disinfection cycles (equivalent to cleaning and disinfecting once a week over the 5-year service life of the meter and lancing device).

Cleaning is the physical removal of organic soil from the meter and lancing device surfaces. By keeping your meter and lancing device clean, you help ensure that it is working properly and that no dirt gets in the device. Cleaning allows for successful, subsequent disinfection.


Disinfection is a process that destroys pathogens, such as viruses and other microorganisms on the meter and lancing device surfaces. By disinfecting your meter and lancing device, you help ensure that no infection is passed when you or others come in contact with your meter and lancing device.

Step	Action
1	Assemble cleaning and disinfecting supplies: Dispatch® Hospital Cleaner Disinfectant Towels with Bleach EPA 56392-8 (available online through major retailers). These Disinfectant Towels contain a 0.65% NaOCl solution and have been shown to be safe for use with the meter and lancing device.
2	<div data-bbox="296 726 1225 855">IMPORTANT: Do not place the meter or lancing device in water or other liquids. Avoid getting dust, dirt, blood, control solution, water, or any other substance in the meter's test strip port and USB port.</div> <p>For cleaning, wipe the outside surfaces of the meter and the meter Home Button or lancing device until they are visibly clean. Clean your meter or lancing device at least once a week when visibly dirty. (See <i>Meter Specification</i> section.)</p>

2

(Continued)

For disinfection, wipe all outside surfaces of your meter and the meter Home Button including the test strip port or lancing device until they are wet in order to pre-clean the meter or the lancing device. After pre-cleaning, use a second wipe to disinfect the meter or lancing device using the same process. Avoid getting bleach solution in the USB port as this may damage the meter. Surfaces must remain wet for a full one (1) minute. Disinfect your meter or lancing device at least once a week. (See *Meter Specification* section.)

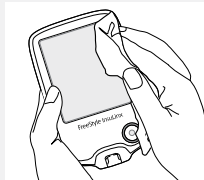

3

Allow the meter or lancing device to air dry completely before performing a blood glucose test.


Note: If you notice any signs of deterioration on the meter or lancing device (such as clouding or crazing on the display of the meter, corroding or eroding of the plastic housing or button, or cracking of plastic housing, display, or button) or if the meter does not turn on or if the lancing device does not work after cleaning and disinfection, discontinue use of the meter or lancing device and contact Customer Service.

4

When finished, thoroughly wash your hands with soap and water. If you require assistance or see signs of deterioration on your meter or lancing device (such as clouding or crazing on the display of the meter, corroding or eroding of the plastic housing or button, or cracking of plastic housing, display, or button), contact Customer Service at 1-800-222-3100.

Troubleshooting

1. Test strip is inserted in the port and nothing happens.


What It Means	What To Do
<ul style="list-style-type: none">• Test strip is inserted incorrectly or not fully inserted into the meter	<p>Insert the test strip as shown.</p> 
<ul style="list-style-type: none">• No batteries are installed• Batteries are installed incorrectly	<p>Install batteries with (+) facing up.</p>
<ul style="list-style-type: none">• Dead batteries	<ol style="list-style-type: none">1. Change batteries. Reset date and time, if needed.2. If problem continues, contact Customer Service.
<ul style="list-style-type: none">• Meter may be plugged into a computer	<p>Unplug the meter from the computer.</p>
<ul style="list-style-type: none">• Problem with test strip	<p>Try a new test strip.</p>
<ul style="list-style-type: none">• Problem with meter• Blood or foreign objects put in the test strip port	<p>Contact Customer Service.</p>

	What It Means	What To Do
2. The test does not start after applying the blood drop.	<ul style="list-style-type: none"> • Blood drop is too small 	<p>Add blood to the same sample area within 60 seconds, or repeat the test using a new test strip and a larger blood drop.</p>
	<ul style="list-style-type: none"> • Problem with test strip • Sample applied after meter turns off 	<ol style="list-style-type: none"> 1. Repeat the test using a new test strip. Wait for 'apply blood to strip' message to appear before applying blood (or 'apply control solution to strip' message if you are applying control solution). 2. If problem continues, contact Customer Service.
	<ul style="list-style-type: none"> • Problem with meter 	<p>Contact Customer Service.</p>

Getting to Know Your FreeStyle Auto-Assist Software

Welcome to FreeStyle Auto-Assist. This software tool allows you to take a more informed role in managing your diabetes. It can help you and your health care professional evaluate needs and options for your diabetes management program.

Note: Your meter must be connected to the computer in order to modify settings or view reports. Reports will be created based on the current data saved on your meter. No data from the meter logbook is saved to your computer.

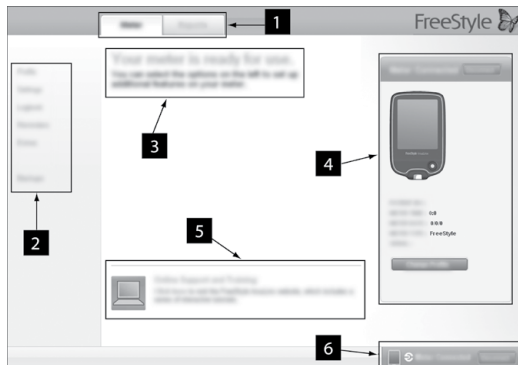

1. **Meter and Report tabs:** Provide access to meter setup features or report options.
2. **Submenu:** Provides access to additional features within each tab.
3. **Message box:** Keeps you posted on software status and any next steps (if needed).
4. **Meter information:** Shows meter details such as time, date, serial number, and patient name/ID.

5. **Online Support and Training:** Provides a link to online training and support.
6. **Meter Status Bar:** Displays meter connection status.

Install the FreeStyle Auto-Assist Software

Note: FreeStyle Auto-Assist is compatible with both PC and Mac operating systems.

Step	Action
1	With your meter off, use the provided micro USB cable to connect your meter's USB port to your computer. If you do not have the micro USB cable provided with the meter, you may use other micro USB cables.
2	The software will start automatically for installation or use. Follow the InstallShield Wizard prompts. Click Next after each entry. Note that you may be asked to supply an administrator password.
3	Click Finish or Close once installation is complete. The program will automatically launch whenever you connect the meter to your computer.

WARNING: To avoid the possibility of electric shock, never perform a blood glucose test while the meter is connected to the computer.

If the installation does not automatically start, follow the steps below to install the software.

Windows Computer Systems (includes Windows 7, Windows Vista and Windows XP)

Step	Action
1	Locate "My Computer" on your desktop. Double-click "My Computer" to open.
2	Locate the "FreeStyle Auto-Assist" icon. Double-click on this icon to open.
3	Locate the "Windows" folder. Double-click on this folder to open.
4	Locate the "Disk1" folder. Double-click on this folder to open.
5	Locate the "Setup application" icon. Double-click on this icon to begin the installation.

Macintosh Computer Systems (OS X, includes Leopard, Snow Leopard and Lion)

Step	Action
1	In the Finder window, select the Device called "FreeStyle Auto-Assist" in the Devices section.
2	Within the FreeStyle Auto-Assist Device, select "Mac".
3	Double-click on the "FreeStyle Auto-Assist.pkg" file.

For additional questions, please contact Customer Service.

FreeStyle Auto-Assist System Requirements

- At minimum, a computer with a USB 2.0 compliant hub running Windows with a single-core 1GHz processor and 512MB RAM or running Mac OS X Leopard with an Intel dual-core 1.83GHz processor and 512MB RAM.
- Compatible operating systems:
 - Windows 7 (32 and 64 bit)
 - Windows Vista (32 and 64 bit)
 - Windows XP (Service Pack 3)
 - Macintosh Leopard, Snow Leopard and Lion (Intel processor)

Change Your Meter Settings

Change clock style, language, or sound preferences.

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Settings .
2	Make your selections: <ul style="list-style-type: none">a. Clock style, click your choice.b. Language, select your choice.c. Sound, click On or Off to change your Notification or Button sounds.
3	Click Save to meter .



Set up Advanced Features for Your Meter Logbook

Set up additional meter logbook features such as the ability to log all your insulin doses and add up to 11 custom notes that can be logged with your blood glucose test results.

Set up the Insulin Log Feature


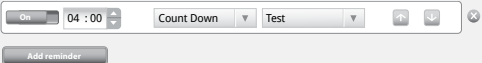
Step	Action
1	Go to the Meter tab. On the left side of the screen, click Logbook .
2	Make your selections: a. The feature is on when you receive the meter. If you want to turn it off, click the button to Off . If you want to turn it back on, click the button to On to record your insulin doses. b. Check the appropriate boxes for the types of insulin you take.
3	Click Save to meter . For more information about adding insulin doses to your meter logbook, refer to the <i>View Logbook</i> section.

Set up Notes for Your Meter Logbook

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Logbook .
2	Click Notes on the Logbook submenu.
3	<p>Make your selections:</p> <ol style="list-style-type: none">Click the button to On to use Notes.The first four notes are: Exercise, Felt ill, Medication, and Felt low. You can choose to accept these; orEdit, change order, delete, or add up to 11 custom notes.<ul style="list-style-type: none">Click Edit note to change the name of a note. Then click Done.Change the order of the notes by using the up and down arrows (this will change the order they appear on the meter).Click the  button next to the note you want to delete.Click Add note to add a custom note. 
4	<p>Click Save to meter.</p> <p>For more information about adding notes to results in your meter logbook, refer to the <i>View Logbook</i> section.</p>


Set up Meter Reminders


Set your meter to remind you when to test, when to take your insulin, or for other alarms. The software allows you to set up to 12 different personal reminders.

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Reminders .
2	Set up to 12 reminders by clicking Add reminder .
3	<p>Set up your reminders:</p> <ol style="list-style-type: none">Click the On/Off button to turn a reminder On or Off.Use the up and down arrows to select the time.Use the drop down menus to select when you want the reminder repeated and the type of reminder. You can also add a custom type of reminder.Click Add to add the reminder.Change the order of the reminders by using the up and down arrows (this will change the order they appear on the meter).Click the  button next to the reminder you want to delete. 
4	<p>Click Save to meter.</p> <p>For more information about reminders on your meter, refer to the <i>Set up and Use Reminders</i> section.</p>

Set up Your Meter Personalization

Change the picture you see when you perform a blood glucose test.

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Extras .
2	Make your selection: <ul style="list-style-type: none">• Click on one of the available pictures to select it. Click Save to meter. You will now see this picture each time you perform a blood glucose test.• If you would like to use a personal picture from your computer, go to Step 3.
3	Click My Picture .
4	Upload a personal picture from your computer. <ol style="list-style-type: none">a. Click Choose.b. Select your picture. Click Open.c. Adjust your picture so that it fits inside the borders of the box. To change the size, use the slider bar. d. Click Set Picture.e. Click Save to meter. For more information about personalizing your meter, refer to the <i>Personalize Your Meter</i> section.

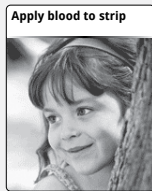

Example
personal picture

Change Rapid-acting Insulin Dose Increments

Set your insulin dose increments to either 1.0 or 0.5 units for use with the Insulin Log feature.

IMPORTANT: It is strongly recommended that you work with your health care professional to determine which insulin dose increment is best for you.

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Extras .
2	Click Dose Increments .
3	Click your choice, 1.0 unit or 0.5 unit.
4	Click Save to meter .

Set up Weekly Messages on Your Meter




Turn on weekly messages. This feature will help you identify patterns in your blood glucose results.




IMPORTANT: Ask your health care professional for your blood glucose target ranges.

Step	Action
1	Go to the Meter tab. On the left side of the screen, click Extras .
2	Turn on Weekly Messages. a. Click Weekly Messages . b. Click the button to On to use this feature. c. Enter your target range.
3	Click Save to meter . For more information about weekly messages on your meter, refer to the <i>Set up and Use Weekly Messages</i> section.

FreeStyle Auto-Assist Reports

The FreeStyle Auto-Assist software allows you to choose from six reports:

Report	What It Does
	<p>The Snapshot report is a general summary of data for the specified date range. It also includes notes that inform you and your health care professional about important trends in blood glucose data by putting the data into words. Up to six notes will be displayed at one time, in order of clinical significance.</p>
	<p>The Modal Day Report shows the daily pattern of blood glucose levels over the specified date range.</p>
	<p>The Logbook Report is a table of blood glucose and insulin values for each day in the specified date range.</p>

Report	What It Does
	<p>The Daily Statistics Report provides an overview of blood glucose and insulin data over the date range and displays it in a series of easy-to-read tables.</p>
	<p>The Meal Event Averages Report compares the before meal and after meal blood glucose levels for the morning, midday, and evening times over the specified date range. This report also shows before and after meal averages, and insulin intake for meals.</p>
	<p>The Meter Settings Report shows current meter and insulin settings.</p>

Set up or Change Report Preferences


The FreeStyle Auto-Assist software allows you to set the following report preferences:

- Report set
- Time frame
- Glucose Targets
(includes hypoglycemia threshold)
- Auto-save options
- Print color
- Quick print

IMPORTANT: Work with your health care professional to understand your report data.

Note: Your blood glucose targets and hypoglycemia threshold preferences will only be used in the reports and will not change any meter settings.

Note to health care professionals: These preferences will be stored on your computer and will be used when creating reports from any meter.

Step	Action
1	Go to the Reports tab. If this is the first time you are creating reports, click  to be guided through the report preferences setup.
2	If you wish to change your report preferences, click Report Preferences on the left side of the screen. Use the submenus to change your report preferences.

Create, View, Save, and Print Reports

Step	Action
1	<p>Go to the Reports tab. On the left side of the screen, click Create Reports.</p> <p>Note: You can change report settings from this screen by clicking Edit on the setting you wish to change. Click Done to save changes. These changes are not stored in Report Preferences.</p>
2	<p>If you would like to preview the reports, click View on Screen. Use the arrows to view the next/previous report page.</p> <p>To save the report as a PDF, click Save.</p> <p>Note: The Adobe Acrobat Reader program must be installed on your computer to save files in Adobe Acrobat PDF format. Go to www.Adobe.com for instructions on downloading this program free over the Internet.</p> <p>Click Print to print your selected reports.</p>

Additional FreeStyle Auto-Assist Features

Feature	What It Does	How To Access
First time Meter Setup	Sets your meter's language, date, and time.	If you have not yet set up your meter, plug it into your computer and click Set up Meter from the Welcome screen.
Export Data	Exports meter data to a text file.	From the menu bar at the top of the screen, click File . From the drop down, click Export data .
Update Software	Installs updates to your FreeStyle Auto-Assist software and meter.	Your FreeStyle Auto-Assist software will alert you when updates are ready.
Backup Meter Files	Saves a backup file containing your current meter settings and preferences.	Go to the Meter tab. On the left side of the screen, click Backups . Click Save backup file .
Restore Meter Files	Replaces your current meter settings with settings from a backup file.	Go to the Meter tab. On the left side of the screen, click Backups . Click Restore Settings .

IMPORTANT: Make sure you choose the proper file to restore. If the incorrect file is chosen, the meter may be set with the wrong settings (such as insulin settings or blood glucose targets).





Meter Specifications

Altitude	Up to 10,000 feet (3,048 meters) above sea level
Assay method	Coulometric electrochemical sensor
Automatic shutoff	Two minutes of inactivity
Battery life	Up to 3000 tests
Calibration	Plasma equivalent glucose values
Cleaning and Disinfection testing	522 cleaning and 522 disinfection cycles (equivalent to cleaning and disinfecting once a week over the 5-year service life of the meter and lancing device)
Hematocrit	15% to 65%
Mean Service Life	5 years
Measurement unit	mg/dL
Meter storage temperature	-4 °F to 140 °F (-20 °C to 60 °C)

Memory	Up to 165 days based on an average of 3 tests and 3 insulin logs per day
Minimum Computer Requirements	System must only be used with EN60950-1 rated computers. Use a USB certified cable.
Operating relative humidity	5% to 90% (non-condensing)
Operating temperature	40 °F to 104 °F (4 °C to 40 °C)
Power source	Three CR 2032 3V lithium batteries, two of which are replaceable
Result range	20 mg/dL to 500 mg/dL
Sample	Fresh capillary whole blood samples
Sample size	0.3 microliter (300 nanoliters)
Size	2.4 in (w) x 3.8 in (l) x 0.6 in (d) 6.0 cm (w) x 9.5 cm (l) x 1.5 cm (d)
Weight	2.1 to 2.3 oz. (59.4 to 65.6 g) including batteries

Appendix — End of Meter Life/ Backup Battery Removal

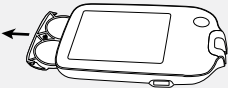Note: Carry out this process only when you no longer intend to use the meter. The meter will no longer operate once opened to remove the backup battery.

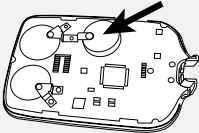Step		Action
1		Remove battery door.
2		Insert small screwdriver blade between case above the USB port.
3		Pry case open with screwdriver. Continue prying around the case until the case separates.
4		Remove backup battery and dispose of battery and meter properly.

References:

- ¹"FDA Public Health Notification: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication" (2010) <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>
- ²"CDC Clinical Reminder: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens" (2010) <http://www.cdc.gov/injectionsafety/Fingerstick-DevicesBGM.html>
- ³American Diabetes Association. Standards of Medical Care in Diabetes—2011. *Diabetes Care* 2011; 34 (Suppl.1) S11-S61.
- ⁴*Diabetes Care*. January 2010, vol. 33, no. Supplement 1, S11-S61.
- ⁵Hirsch, I. (2005). Glycemic Variability: It's Not Just About A1C Anymore! *Diabetes Technology and Therapeutics*, 7 (5), 780-783.
- ⁶*Mayo Clinic Proceedings* February 2007, vol. 82, no. 2, 229-236.
- ⁷Pearson J, Bergenstal R. Fine-Tuning Control: Pattern Management Versus Supplementation: View 1: Pattern Management: an Essential Component of Effective Insulin Management. *Diabetes Spectr.* April 1, 2001;14(2) 75-78.

Manufacturer:
Abbott Diabetes Care Inc.
1360 South Loop Road
Alameda, CA 94502

Customer Service is available
24 hours a day,
7 days a week:
1-800-222-3100
www.MyFreeStyle.com

FreeStyle and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions. All other trademarks are the property of their respective owners.

This product(s) and/or its use are protected by one or more of the following patents: US6,071,391; US6,120,676; US6,143,164; US6,156,173; US6,299,757; US6,338,790; US6,503,381; US6,591,125; US6,616,819; US6,618,934; US6,676,816; US6,749,740; US6,893,545; US6,942,518; US7,058,437; US7,615,637; US7,713,406; US7,740,581; US7,802,467; US7,866,026; US7,895,740; US7,901,554; US7,918,012; US7,918,988; US7,996,988; US8,033,162; US5,822,715; US6,379,301; US7,167,818; US7,392,167; US7,941,308; US5,918,603; US5,601,435; US6,144,837; USD611,8545; USD613,1905; USD615,8845; USD646,7875; USD648,6405; EP2069771; EP1119637; EP1145000; EP0958495, CA2346415; CA2358993; CA2351796; CA2423837; and CA121137. Additional patents may be issued and/or pending.

